



By far the most humane

INSTRUCTION MANUAL

JABSTICK

COMPLETE AUTOMATIC TRANQUILIZING
& MEDICATION SYSTEM (CATS)


Professional and humane equipment
for immobilization and treatment



Warning

To avoid damage to the internal gas spring, the Jabstick must never be charged without the nylon syringe in the aluminium cartridge.

When the Jab-Stick is not in use the gas spring must not be compressed.

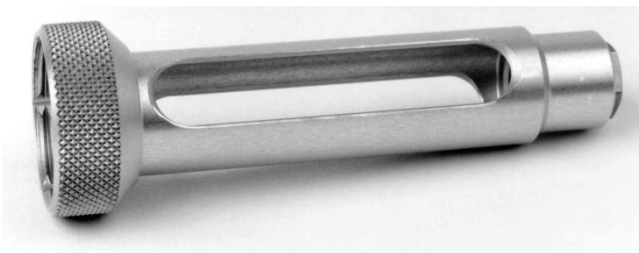

1.

Remove the aluminium cartridge.



2.


Ensure that the hypodermic needle is mounted securely. Do not use hypodermic needles below 1,8 mm in diameter.

After use clean the nylon syringe. If the plunger moves sluggishly apply 1-2 drops of silicon oil within the barrel and the syringe will work as new.




3.

The nylon syringe is inserted in the aluminium cartridge.

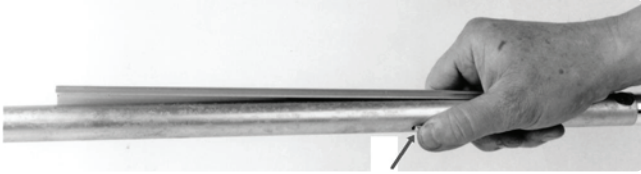

4.


Unlock the Jabstick and the cocking arm will release.

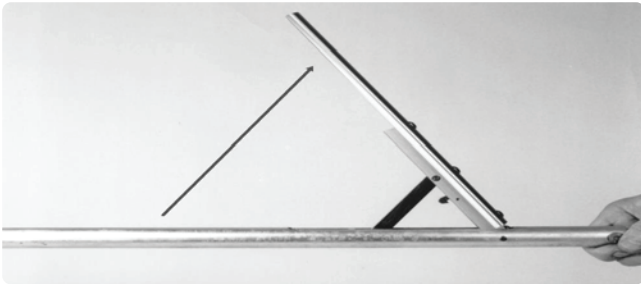

5.


Important:

The arm must be extended to the limit of its travel.

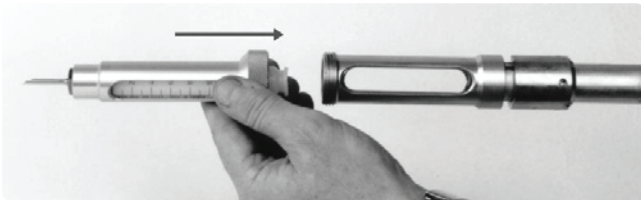

6.


Screw the aluminium cartridge containing the syringe onto the Jabstick.

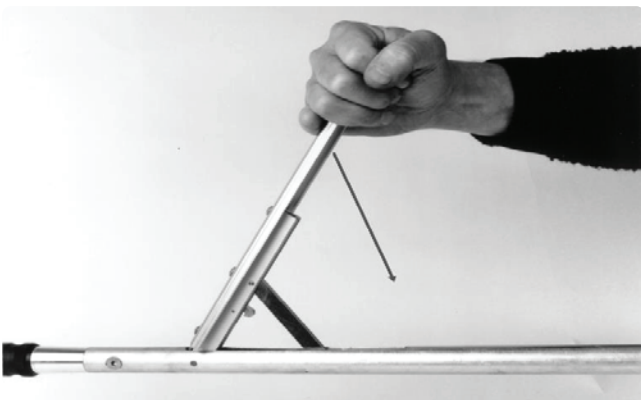

7.


Compress the gas spring by lowering the cocking arm. Some resistance will be encountered.

The arm will lock automatically when it reaches the body of the Jabstick.

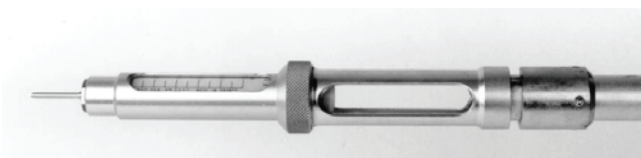

8.

Fill with liquid 1-12 ml.

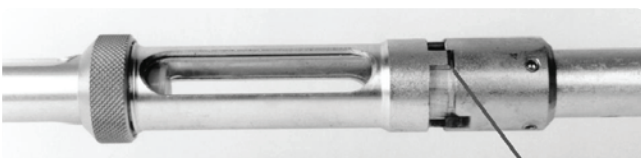

9.

Locked.

The Jabstick is shown in the locked position.




10.

Unlocked.


The Jabstick is shown in the unlocked position.

Extending the handle

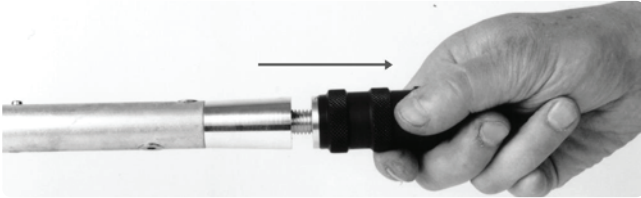


1.

To attach the telescopic extension first remove the plastic handgrip.

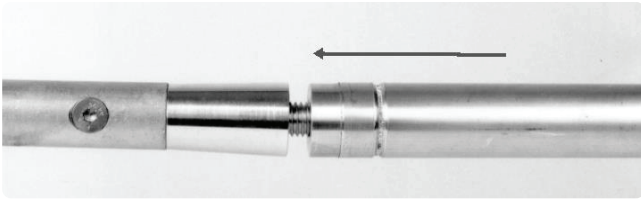

2.

Attach the extension handle.

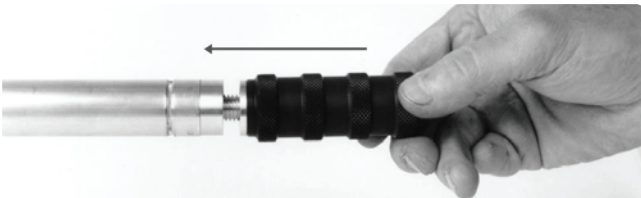

3.

Fit the plastic handgrip on the extension.